### OPED bewegt

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### OPED gets you moving

We at OPED are experts in medical technology. We develop all our orthopedic rehabilitation products in-house and manufacture them in Germany. With free comprehensive follow-up treatment concepts, we accompany the patient from the beginning of medical care until full recovery. Our goal: A mobile patient. [www.oped.de](http://www.oped.de)

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### OPED Australia Pty Ltd

4/42 Greenacre Rd  
Wollongong NSW 2500  
Australia  
Phone +61(0)2 4226 5880  
Fax +61(0)2 4226 5881  
contact@oped-australia.com.au  
www.oped.com.au

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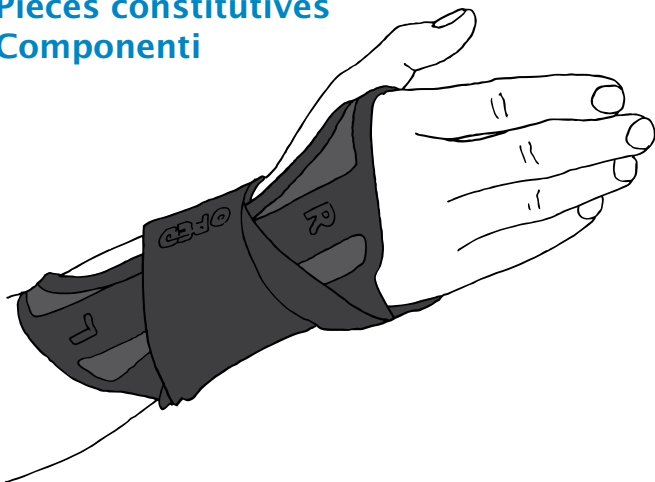
**E** Wrist bandage for use on both sides. Reinforced both on the back of the hand (dorsal) and at the palm (volar).

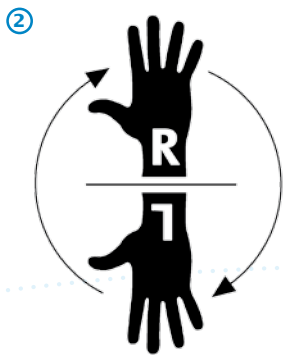
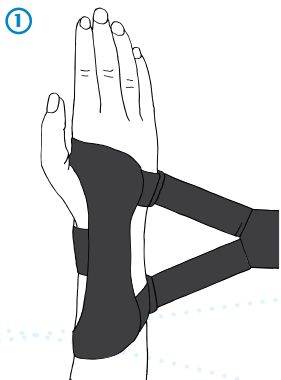
**Delivery is made for application on the right hand!**

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### **E** Left/right

- The printed L (left hand) and R (right hand) indicates the application position of SUPROhand.
- For application to the right hand (marked R), the thumb of the hand points towards the left.
- For application to the left hand (marked L), the thumb of the hand points towards the right.

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### **E** Turning the reinforcing splint to adapt it from application on the right to application on the left hand

- SUPROhand can be used for the left or right hand simply by turning the demi-gauntlet bandage around by 180 degrees. Depending on the given application side, the supplied reinforcing splint for the inner hand side must be inserted accordingly.
- Pull the reinforcing splint out of the slot integrated in the bandage and turn the splint.
- Note: The short, curved side points towards the finger tips when inserted correctly.
- When inserting the reinforcing splint ensure that the short curved side always points towards the finger tips.
- Guide the splint into the slot integrated in the inner side of the bandage accordingly. Afterwards, ensure that the splint is enclosed by fabric at both ends.

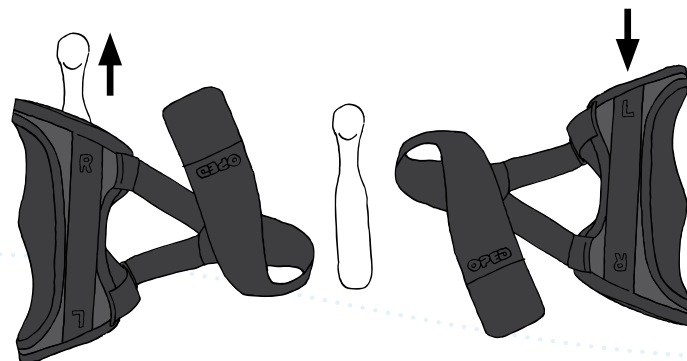
## Anlegevorgang Application procedure Processus de mise en place Procedura applicativa

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- The reinforcing splint for the back of the hand is already integrated. Its position does not have to be changed when the bandage is applied.

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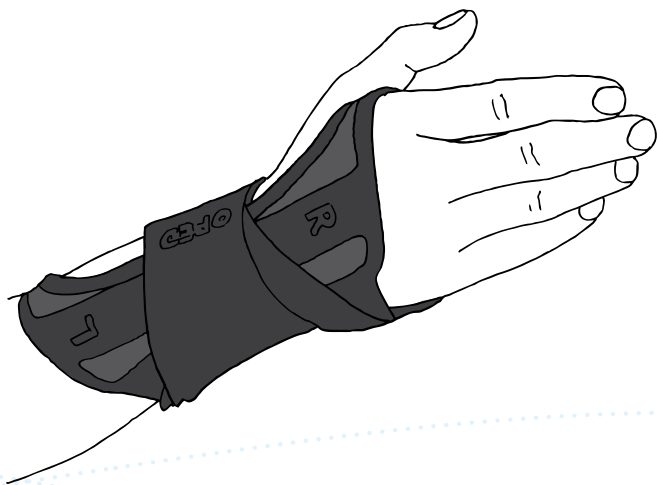
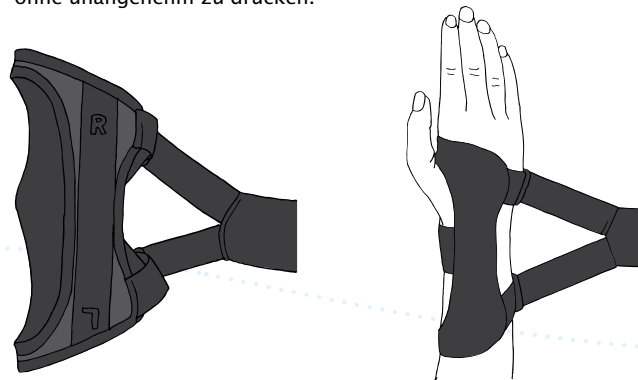
### **E** Applying SUPROhand

- Insert the hand in the SUPROhand demi-gauntlet bandage (see photo for thumb position).
- Pull the elastic-taped Velcro® fastener with the required tension over the lower arm towards the body and around the wrist. First attach it to the facing hook tape on the lower arm side before final encircled fastening.
- If necessary, the Velcro® fastener can be released from the hook tape for re-adjustment and final fastening.
- SUPROhand has been correctly applied when, in the neutral position (outstretched hand), the pre-shaped volar reinforcing splint formfits with the lower arm and especially with the transition from the wrist to the metacarpus.
- If necessary, the reinforcing splint at the palm can be individually adapted to the anatomical shape of the hand and lower arm or according to the doctor's instructions by bending it.
- SUPROhand should be firmly seated on the hand to achieve the required stabilizing effect, but without causing any unpleasant pressure.

## Anlegevorgang Application procedure Processus de mise en place Procedura applicativa

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## Useful Details

### Cleaning and maintenance

Wash SUPROhand in lukewarm water with a cold-water detergent and air dry. Do not place on a radiator. Remove the reinforcing splints from the bandage before washing. Close the Velcro® fasteners before washing as this will lengthen their useful life and avoid damage to other laundry.

### Function

SUPROhand is suitable for both hands, i.e. it can be applied to the left or right hand. The volar (inner side of the hand) and dorsal (back of the hand) reinforcement splint, as well as the circular stabilization tape around the wrist, ensure optimal partial immobilization of the wrist. The carpus is fixated, while the thumb and index finger are freely moveable. As a result of the carefully designed cut of the anatomical blank, the SUPROhand offers optimal fit and function for both the left and right hand. Moreover, the open character of the wrist bandage material makes it pleasant and comfortable to wear.

### Indications

- Post-operative treatment of radius fractures
- Arthrosis of the wrist
- After distortions
- Tendovaginitis
- Functional postoperative after-treatment
- Overstrained arthropathies
- Interim care after plaster bandages have been removed
- Conservative treatment of the initial stages of aseptic necrosis of the carpal bones
- Wrist-drop

### Contra-indications

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated
- Impaired circulation or lymphatic soft tissue swelling
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage)

So far not consequential incompatibilities or allergic reactions have been reported.

### Side-effects

With proper use and proper fitting, so far there have been no reports of serious general side-effects. Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restricting consistently shaped body fitting.

### Important

The product is to be used by one patient only. Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from productliability. You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you. For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions. Please dispose of your used packaging at the local recycling depot.

### Technische Daten / Parameter

#### Materials used

Textile fibers (100% polyester); metal splints (aluminum)

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