OPED bewegt

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OPED gets you moving

We at OPED are experts in medical technology.
We develop all our orthopedic rehabilitation
products in-house and manufacture them in
Germany. With free comprehensive follow-up
treatment concepts, we accompany the patient
from the beginning of medical care until full
recovery. Our goal: A mobile patient.
www.oped.de

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Introduction

The Pollex is specifically designed to stabilize and immo-
bilize the the base joint of the thumb (carpometacarpal
or CMC joint). The Pollex can also be used in conjunction
with the VACOhand wrist brace to create cast-like stabi-
lization for the thumb and wrist. The ease with which the
Pollex can be handled, applied and removed, as well as
its range of motion ability makes it a simple, highly
functional treatment option.

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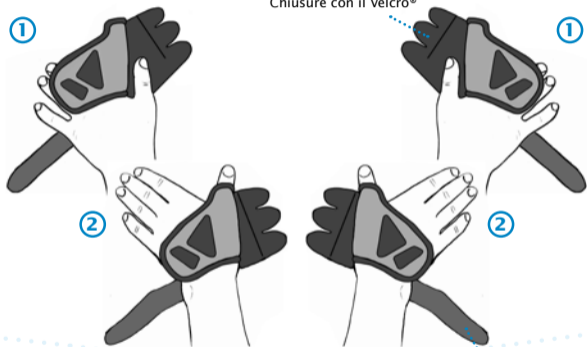
E Put on sleeve
(this applies only when being used with VACOhand)

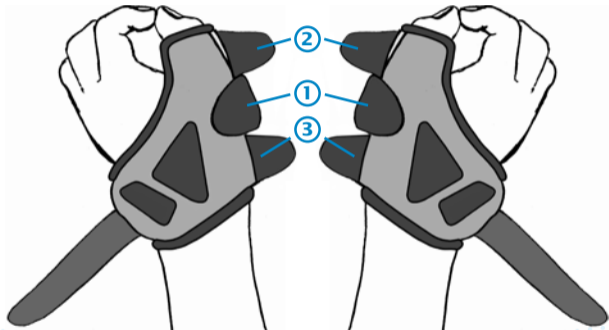
- Open all the Velcro® closures ①
- Insert thumb into position shown in illustration ②

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E

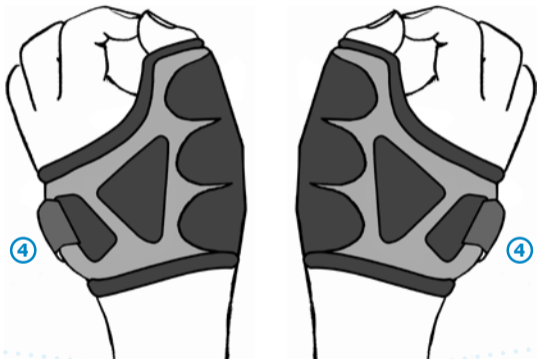
- Bring the tips of the thumb and index fingers together
- Close the Velcro® tabs beginning with medial tab, then the upper and lower tabs respectively (see illustration).

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E

- Close wrist Velcro® wrist strap ④ . Use scissors to trim excess.
- Check all velcro closures are sealed correctly.
- Readjust if necessary ④

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Useful Details

Cleaning and Care

- Soak the Pollex in lukewarm, soapy water to clean
-

Product features

- 2 sizes: S and M/L
each available for the left or right hand
-

Indications/applications

- Game-keeper/ski thumb
 - Thumb fractures (post-operative)
 - Soft-tissue injuries
 - Scaphoid fractures (Post-operative, when used together with VACOhand)
-

Warranty

The product warranties comply with all legal regulations.

Problem & solution

Pain and swelling

- If readjustment or reapplication does not provide relief, contact your treating physician at once.
-

Technical data/dimensions

- Material used
Hard shells (PE); covers (PE); straps (PA/PP)
 - Weight 15,5 g size S; 25,5 g size M/L
Dimensions 11x10 cm size S; 14x13 cm size M/L
-

Safety notes/contraindications

- Always ensure the hard shell and soft components are free of any damage.
- Always consult your physician before using Pollex
- No contraindications are known.

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